

AXIS EO GAS METAL CARTRIDGES



KORDON

Tıp Sağlık Araç ve Gereçleri

Manufacturer: KORDON TIP SAGLIK ARAC GERECLERI
PROJ. MUH. ITH.SAN. VE TIC.LTD.STI.

10006/1 No:43 A.O.S.B CIGLI IZMIR TURKIYE

www.kordontip.com * info@kordontip.com

Tel: +90232.348.60.60 * Fax:+90232.348.60.90





Code	Active Ingredient	Diameter x Height	Net Content
AL 7	100% Ethylene Oxide	22 x 58 mm	11g ±10%
AL 25		22 x 75 mm	25g ±10%
AL 67		35 x 105 mm	67g ±10%
AL 100		35 x 165 mm	22g ±10%
AX 67		38 x 105 mm	67g ±10%
AX 100		38 x 165 mm	100g ±10%
AX 127		38 x 165 mm	127g ±10%
AX 134		38 x 165 mm	134g ±10%
AQ 120		50 x 133 mm	120g ±10%
AQ 130		50 x 133 mm	130g ±10%
AQ 170		45 x 178 mm	170g ±10%
AQ 200		50 x 174 mm	200g ±10%
AQ 230		50 x 174 mm	230g ±10%

Compatible with;
 Belimed® – 3M Sterivac® – Amsco® – Hogner® – Del Giudice®
 MMM®– Quetzal® – Mazden® – BioMedica® – Mazzarotti®
 EO Gas Sterilizers.

Axis provides 100% Pure Ethylene Oxide Gas in aluminum cartridges with controlled quantities. This system provides safety in the sterilization areas, as the cartridge can only be punctured inside the sterilizer chamber from its metal cap. These cartridges are packed under strict safety regulations to meet the requirements of the sterilizers in the market. CE, ISO 13485 and ISO 9001 certified.

How does it work?

Prior to starting a sterilization cycle, the operator inserts the EO cartridge into the cartridge holder. After the completion of the sterilization and the aeration cycle, the empty cartridge can be simply removed from the cartridge holder and discarded with waste material. For running sterilizer please follow the sterilizer manufacturer instructions. Use Axis metal cartridges depend on the sterilizer manufacturer's instructions.

Generally:

1. Open the sterilizer door.
2. Insert the cartridge into the cartridge slot.
3. Close the sterilizer door.
4. Start the program.



Axis AX-1800 EO Gas Alarm

Applications include chemical and medicine (hospital sterilization facilities) and other areas where leakage or generation of ethylene oxide is possible. Measures the Ethylene Oxide concentration in air at the 0-99 ppm range

Kordon Tip Recommends



Features

- 100% Pure Ethylene Oxide.
- Bactericide, Viricidal and Fungicide Action.
- Single-use and Airtight.
- CFC Free (Freon) that does not affect the Ozone Layer.
- Safe for plant personnel.
- No special disposal is required

Order from info@kordontip.com / sales@kordontip.com
 Tel: +90232.348.60.60 * Fax: +90232.348.60.90